



STRAVA SPLINT BATH IN-LINE FILTER CLEANING

Your Strava Splint bath is fitted with a small in-line filter which protects the circulation pump against small impurities and debris which can accumulate over time and clog the pump.

Please remove and clean the in-line filter every time you drain and clean the bath, or more often if needed.

1. Evacuate the water.
2. Turn power switch off.
3. Using the supplied Allen key, loosen the filter.
4. Wiggle the filter off of the thermocouple.
5. Clean all debris from the screen (hint: run under water).
6. Fit the cleaned filter back on the thermocouple by wiggling it into place (screw heads facing out).
7. Using the supplied Allen key, tighten the filter insuring that it is properly aligned and close to the hole.

Re-ordering information:
REF 300.217 In-Line Filter

Manufactured by:
STRAVA SOLUTIONS
2733 Kanasita Drive #125
Hixson, TN 37343 USA



PART NUMBER: REF 500.702

